



easywrap[®]

COMPRESSION MADE EASY



CONTENTS



Introduction

Page 4 ►



Features & Benefits

Page 6 ►



easywrap Leg

Page 18 ►



easywrap Foot

Page 20 ►



easywrap Thigh

Page 22 ►



easywrap Liners

Page 24 ►



easywrap Arm

Page 28 ►



easywrap Hand

Page 30 ►

INTRODUCTION

Haddenham easywrap is a UK patented compression wrap system for the treatment of lymphoedema, chronic oedema and various other swelling related conditions.

easywrap garments consist of overlapping, single-layer textile bands that are worn over a protective antimicrobial liner.

Its low profile design means that it can be worn comfortably under most everyday clothing and footwear...



...even jeans!

TWO COMPRESSION CLASSES

easywrap strong 30-40mmHg provides similar stretch to a traditional short-stretch bandage, with excellent static stiffness performance.

OR

easywrap light 20-30mmHg is a more stretchy fabric suitable for mild to moderate compression needs and palliative care.



The mechanically limited stretch of the bands gives them an easy to feel “lock-out” or “end stretch”, which makes the bands easy to apply at the correct tension.

COMPRESSION WRAP SYSTEM

easywrap lower limb is comprised of a three part garment system, designed to be used either individually or in combination.





WITH LINERS INCLUDED

Worn beneath the wrap system; antimicrobial liners form a barrier between the limb and the easywrap that absorbs sweat and traps odours, reducing the need to wash the easywrap. Antimicrobial liners are available in both A-g and A-d lengths.



easywrap FUSION
liners offer compression
at the foot and half compression
at the ankle, further expanding easywrap's
combination options.

LOW PROFILE DESIGN

easywrap's low profile design means that it can be worn comfortably under most everyday clothing & footwear.



PATENTED KNEE SECTION

An easy to apply three band system.
Included with easywrap **THIGH**.

Elliptical cut with spacer foam pad
increases comfort and mobility whilst
reducing pinching behind the knee



EASY TO APPLY

Easy-grab smart fasteners that only stick where they're supposed to.

Easy to feel "lock-out" or end-stretch makes them easy to apply at the correct tension so that you can be confident that you're getting the correct level of compression every time.




NO LOSS OF DEXTERITY

Also available for treatment of the upper limb, easywrap arm and hand garments suffer no loss of dexterity while maintaining compression.




ENCOURAGES MOBILITY

High working pressures mean that the more active you are, the better your lymphatic system will work to reduce oedema.

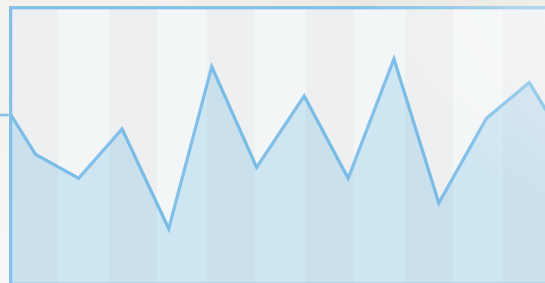

MAINTAINS LIMB REDUCTION

**...and when you're less active
easywrap's lower resting pressures
maintain limb reduction.**

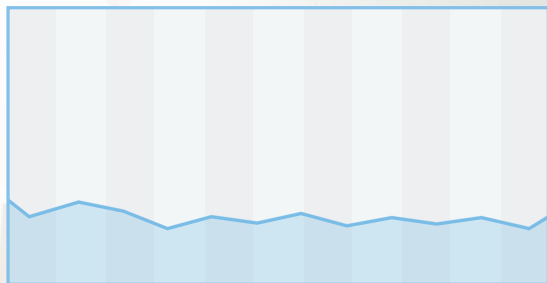



easywrap allows you to have more control over the swelling of your legs, which I think is very important. I can see a visible difference in my leg after wearing this which is exactly the result I think everyone with lymphoedema wants.

- Mrs. W, Lymphoedema patient

This is very easy to put on, comfortable and really helpful for my lymphoedema - especially when the swelling is particularly uncomfortable (e.g. in hot weather).

- Carol P, Lymphoedema patient

“Patients who have previously been non concordant with treatment, whether that be due to the psychosocial aspects of bandages or the inability to apply compression garments have all experienced and increase in quality of life when using easywrap.”

“Evidence suggested in the literature review and clinical case studies demonstrates that velcro compression wraps may be safer to apply than short stretch bandages.”

Download
clinical case
studies and article
www.easywrap.info

British Journal of
Community Nursing

PRODUCT FOCUS

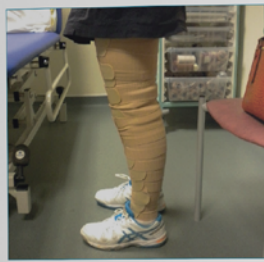


Figure 7. Mrs H, Ready to play tennis wearing her easywrap

of 507ml (11.5%). The only change to Mrs H's self-care regime had been switching from ad hoc wear of Juxta-Fit to more regular wear of easywrap, so that a direct comparison could be made.

Mrs H has stated:

“Overall I am very happy with easywrap, my top

ness in reducing limb mobility, stinging, ease of putting on, ease of getting off. I am now confident in playing tennis (Figure 7) with light sports leggings over, and because I am exercising with compression, I feel it helps the lymphatic system in my affected leg. I am now in the habit of wearing easywrap at every opportunity. After taking it off I will go out socially in short dresses as I feel my affected limb is not so noticeable. I feel it is much better than Juxta-Fit. Being able to play tennis and wearing for gardening means I was able to maintain compression, exercise and reduce any increase in volume.”

Case study 2: Mrs S

Mrs S started to notice her legs were bigger than others aged 11 years old:

“My doctors never knew what it was, I was prescribed slimming pills aged 18 years old but didn't like the side effects and my legs never changed.”

Mr S was referred to the lymphoedema clinic and assessed when she was 26 years old, where she was prescribed compression tights. Her legs became bigger following pregnancy in 2008 and then again in 2010, during, by which time she wore maternity compression throughout. In 2011, Mrs S had a lymphoedema scan, which confirmed slight lymphoedema in the right leg but mostly lipodema overall.

the bandages, they kept unravelling and they were so big that I had to wear ‘size 8 crocs’, not a good look when you are a size 6 shoe.”

Mrs S found bandaging challenging in that they did not stay in place, so she decided to swap them for easywrap. *“Wow... What a difference, I put on the foot/ lower leg easywrap and they were so much more comfortable, stayed secure and I could wear normal footwear on top. I did believe this has had massive improvement to my swelling/recovery. I have worn the wraps daily for 6 weeks now and I still sleep in compression but I've gone back to wearing my Haidenhelm Vini compression. Hanse Klinik do advise that it will take up to 4 months for swelling to go down, but I think my legs look great already—only 6 weeks and 2 days after surgery! Now I feel completely amazing, I'm the happiest I've ever been in my life. I feel like a proper girl for the first time in my life, I have wearing dresses and skirts, my legs feel light and free. I'm happy wearing compression, I like the support it gives me and it's a bit like a comfort blanket. I continue to swim, attending local pools and do not feel conscious about people looking at my legs! My next aim is to join a netball team, as I have not worn a netball skirt since I was at school because I was so self-conscious of my legs, but now I can wear a skirt with pride and show off my stunning legs I've*



Figure 9. Mrs W wearing easywrap

my lymphoedema by bandaging my leg for me regularly and massages my leg, hips, stomach and back every day.”

During 2016, Mrs W attended LymphVision for mapping, which confirmed dermal re-rolling and backflow in the affected leg. It also confirmed that lymph was draining into

COST EFFECTIVE AND TIME SAVING

78 Hours
Nursing Time

7 Hours
Nursing Time

Bandaging

78 Appointments

3x 1 hour appointments per week

Cost of Bandages

£1,935.96

Cost of Nursing Time

£1,872.00

Total Cost

£3,807.96

easywrap

13 Appointments

1x 0.5 hour appointment per week

Cost of easywrap

£292.92

Cost of Nursing Time

£312.00

Total Cost

£604.92

Cost comparison based over 6 months, treating bilateral legs and does not include travel time, primary wound dressings or mileage expenses.

Cost comparisons based on those used in: [https://hadhealth.com/assets/articles/BJCN_24\(4\)_Haddenham_PF_web.pdf](https://hadhealth.com/assets/articles/BJCN_24(4)_Haddenham_PF_web.pdf)

*prices shown based on 2019 Drug Tariff pricing.



LEG

Circumference cm

XS

S

M

L

XL

C WIDEST POINT OF CALF

38-45

44-52

50-60

55-65

60-70

b ANKLE

23-27

28-32

33-38

39-44

45-52

Length cm


REGULAR

TALL

b-d ANKLE TO BELOW KNEE

30-34

34-38

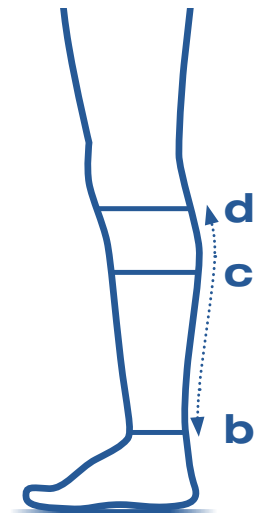

light

20-30 mmHg

REGULAR

TALL

XS	400-4917	400-4966
----	----------	----------

S	400-4925	400-4974
---	----------	----------

M	400-4933	400-4982
---	----------	----------

L	400-4941	400-4990
---	----------	----------

XL	400-4958	400-5005
----	----------	----------

XS	408-9397	408-9355
----	----------	----------

S	408-9405	408-9389
---	----------	----------

M	408-9413	408-9363
---	----------	----------

L	408-9421	408-9348
---	----------	----------

XL	408-9439	408-9314
----	----------	----------

strong

30-40 mmHg

REGULAR

TALL

400-5518	400-5567
----------	----------

400-5526	400-5575
----------	----------

400-5534	400-5583
----------	----------

400-5542	400-5591
----------	----------

400-5559	400-5609
----------	----------

408-9009	408-8951
----------	----------

408-8985	408-8969
----------	----------

408-8993	408-8944
----------	----------

408-9025	408-8977
----------	----------

408-9041	408-8928
----------	----------



FOOT

Circumference cm

XS

S

M

L

XL

a¹ MID-FOOT

22-24

25-27

28-30

31-34

35-40

Length cm

a-heel

REGULAR

14.5-16.5

16-18

17.5-19.5

19-21

21-23

LONG


16.5-18.5

18-20

19.5-21.5

21-23

23-25

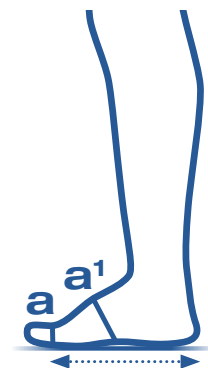

light

20-30 mmHg

REGULAR

LONG

XS	400-4867	400-4818
S	400-4875	400-4826
M	400-4883	400-4834
L	400-4891	400-4842
XL	400-4909	400-4859
XS	408-9553	408-9447
S	408-9546	408-9454
M	408-9538	408-9488
L	408-9520	408-9496
XL	408-9512	408-9504

strong

30-40 mmHg

REGULAR

LONG

400-5468	400-5419
400-5476	400-5427
400-5484	400-5435
400-5492	400-5443
400-5500	400-5450
409-0064	409-0023
409-0049	409-0056
409-0072	409-0031
408-9090	408-9074
408-9116	408-9082



THIGH WITH KNEE

Circumference cm

XS

S

M

L

XL

g

JUST BELOW GROIN

60-70

65-75

70-80

75-90

85-100

e¹

BOTTOM OF THE THIGH

47-52

53-59

60-66

67-73

74-80

Length cm

SHORT

REGULAR

TALL


e¹-g

THIGH TO GROIN

16-20

20-24

25-30

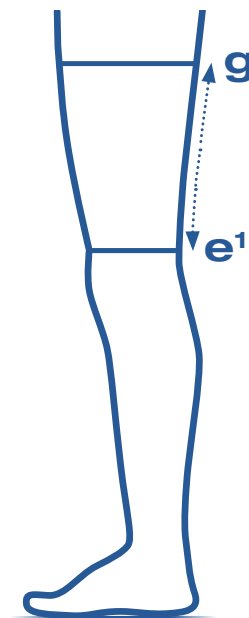

light

20-30 mmHg

	SHORT	REGULAR	LONG
XS	400-5062	400-5013	400-5112
S	400-5070	400-5021	400-5120
M	400-5088	400-5039	400-5138
L	400-5096	400-5047	400-5146
XL	400-5104	400-5054	400-9816
XS	408-9223	408-9231	408-9124
S	408-9215	408-9256	408-9132
M	408-9207	408-9272	408-9140
L	408-9181	408-9280	408-9165
XL	408-9199	408-9298	408-9173

strong

30-40 mmHg

	SHORT	REGULAR	LONG
	400-5666	400-5617	400-5716
	400-5674	400-5625	400-5724
	400-5682	400-5633	400-5732
	400-5690	400-5641	400-5740
	400-5708	400-5658	400-5757
	408-8886	408-8779	408-8712
	408-8910	408-8795	408-8746
	408-8902	408-8811	408-8753
	408-8845	408-8787	408-8738
	408-8860	408-8837	408-8704



A-d LINERS

Suitable for use with easywrap foot and leg garments. Please select the appropriate size for your easywrap garments.

COLOUR

XS / S / M

L / XL



SAND

E-L-AD

E-L-ADW



BLACK

EB-L-AD

EB-L-ADW



A-g LINERS

Suitable for use with easywrap foot, leg and thigh garments. Please select the appropriate size for your easywrap garments.

COLOUR

XS / S / M

L / XL



SAND

E-L-AG

E-L-AGW



BLACK

EB-L-AG

EB-L-AGW






FUSION

easywrap fusion is a liner and foot piece in one, and makes the perfect partner for an easywrap leg piece. The liner provides compression to the foot, and half-compression at the ankle, offering a low-profile, smart alternative to regular foot wraps.

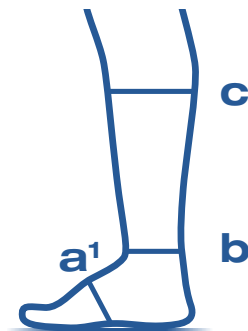

Circumference cm	XS	S	M	L	XL
c WIDEST CALF	≤ 45	≤ 50	≤ 60	≤ 65	≤ 70
b ANKLE	19-24	25-29	30-34	35-40	41-46
a¹ MID-FOOT	19-24	25-29	30-34	35-40	41-46



COLOUR	COMPRESSION	XS	S	M	L	XL
 SAND	Light 20-30mmHg	415-3110	415-3128	415-3136	415-3144	415-3151
	Strong 30-40mmHg	415-3243	415-3235	415-3276	415-3268	415-3250
 BLACK	Light 20-30mmHg	415-3219	415-3185	415-3193	415-3201	415-3227
	Strong 30-40mmHg	415-3284	415-3292	415-3318	415-3326	415-3300

ARM

Circumference cm

S

M

L

XL

g AXILLA

22-31

29-39

32-45

36-50

e ELBOW CREASE

20-27

25-34

30-40

32-43

c WRIST CREASE

14-18

16-21

19-25

19-25

Length cm

SHORT

REGULAR


TALL

c-g AXILLA TO WRIST

40-44

44-48

48-52

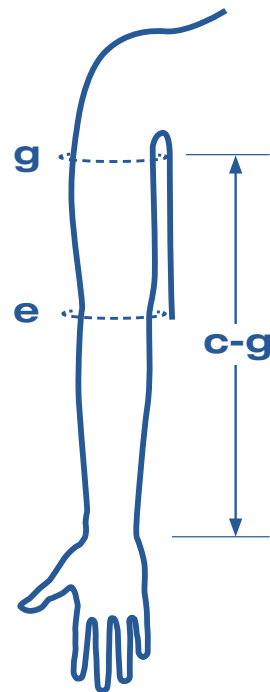

left

20-30 mmHg

	SHORT	REGULAR	LONG
S	400-5187	400-5245	400-5302
M	400-5195	400-5252	400-5310
L	400-5203	400-5260	400-5328
XL	405-8236	405-8194	405-8186
S	408-8373	408-8290	408-8241
M	408-8381	408-8308	408-8225
L	408-8399	408-8316	408-8233
XL	408-8407	408-8324	408-8217

right

20-30 mmHg

	SHORT	REGULAR	LONG
	400-5153	400-5211	400-5278
	400-5161	400-5229	400-5286
	400-5179	400-5237	400-5294
	405-8202	405-8210	405-8228
	408-8415	408-8332	408-8258
	408-8423	408-8340	408-8266
	408-8431	408-8357	408-8274
	408-8449	408-8365	408-8282



HAND

Circumference cm

XS

S

M

L

b PALM AT FOLD OF THUMB

<19

19-21

21-23

>24

a PALM AT BASE OF FINGERS

<18

18-20

20-22

>22



left

20-30 mmHg

REGULAR

XS 400-5377

S 400-5385

M 400-5393

L 400-5401

XS 408-9603

S 408-9637

M 408-9629

L 408-9611

right

20-30 mmHg

REGULAR

400-5369

400-5336

400-5344

400-5351

408-9595

408-9587

408-9579

408-9561



Tel: 01844 208842

Email: sales@hadhealth.com

Website: www.hadhealth.com