### HADDENHAM

## COMPRESSION MADE EASY

## CONTENTS



Introduction Page 4 ►



easywrap Thigh Page 22 ►



Features & Benefits Page 6 ►



easywrap Liners Page 24 ►



easywrap Leg Page 18 ►



easywrap Arm Page 28 ►



easywrap Foot Page 20 ►



easywrap Hand Page 30 ►

## INTRODUCTION

Haddenham easywrap is a UK patented compression wrap system for the treatment of lymphoedema, chronic oedema and various other swelling related conditions.

easywrap garments consist of overlapping, single-layer textile bands that are worn over a protective liner.

Its low profile design means that it can be worn comfortably under most everyday clothing and footwear...

## ...even jeans!

## **TWO COMPRESSION CLASSES**

easywrap strong 30-40mmHg provides similar stretch to a traditional short-stretch bandage, with excellent static stiffness performance.

#### OR

easywrap light 20-30mmHg is a more stretchy fabric suitable for mild to moderate compression needs and palliative care.

The mechanically limited stretch of the bands gives them an easy to feel "lock-out" or "end stretch", which makes the bands easy to apply at the correct tension.

## COMPRESSION WRAP SYSTEM

easywrap lower limb is comprised of a three part garment system, designed to be used either individually or in combination.





## WITH LINERS INCLUDED

Worn beneath the wrap system; liners form a barrier between the limb and the easywrap that absorbs sweat and traps odours, reducing the need to wash the easywrap. Liners are available in both A-g and A-d lengths.

A-d LIA

easywrap FUSION liners offer compression at the foot and half compression at the ankle, further expanding easywrap's combination options.

A-glin

FUSIO

## LOW PROFILE DESIGN

easywrap's low profile design means that it can be worn comfortably under most everyday clothing & footwear.

## PATENTED KNEE SECTION

An easy to apply three band system. Included with easywrap THIGH.

Elliptical cut with spacer foam pad increases comfort and mobility whilst reducing pinching behind the knee





## EASY TO APPLY

Easy-grab smart fasteners that only stick where they're supposed to.

Easy to feel "lock-out" or end-stretch makes them easy to apply at the correct tension so that you can be confident that you're getting the correct level of compression every time.

## NO LOSS OF DEXTERITY

Also available for treatment of the upper limb, easywrap arm and hand garments suffer no loss of dexterity while maintaining compression.

easywrap® ARM and HAND worn with a Microfine® Glove



## ENCOURAGES MOBILITY

High working pressures mean that the more active you are, the better your lymphatic system will work to reduce oedema.



## MAINTAINS LIMB REDUCTION

...and when you're less active easywrap's lower resting pressures maintain limb reduction. easywrap allows you to have more control over the swelling of your legs, which I think is very important. I can see a visible difference in my leg after wearing this which is exactly the result I think everyone with lymphoedema wants.

- Mrs. W, Lymphoedema patient

This is very easy to put on, comfortable and really helpful for my lymphoedema - especially when the swelling is particularly uncomfortable (e.g. in hot weather).

- Carol P, Lymphoedema patient

66 Patients who have previously been non concordant with treatment, whether that be due to the psychosocial aspects of bandages or the inability to apply compression garments have all experienced and increase in quality of life when using easywrap.

66 Evidence suggested in the literature review and clinical case studies demonstrates that velcro compression wraps may be safer to apply than short stretch bandages. 99

### Download clinical case studies and article

### www.easywrap.info

#### PRODUCT FOCUS



Figure 7. Mrs H, Ready to play tennis wearing her easywrap

of 507nl (11.5%). The only change to Mis HV self-areregime had been switching from ad hoc wear of Joxtafit to more regular wear of easywrap, so that a direct comparison could be made. Mrs H has wrand

"Overall I am very happy with easywrap, my top

ners in reducting limb, middly, string, zur of patting on, care of getting off. I can while high space in playing towns (Figure 7) as that high space legging over, and browner 1 am exercising with compression, Jelie 1 is help as hypothesis expression ary affectable leg. I can some in the habit of overring exercises and the second in the shalt of overring exercises and the second in the shalt of overring officed limb. I can see noticed by fire shalts per of I will go and weaking for gradening models. Used in your better than hasta-Fit. Being while to play tensis and weaking for gradening models. I can also maintain compression, exceeded and reduce any increases in protonom?

#### Case study 2: Mrs S

Mrs S started to notice her legs were bigger than other aged 11 years old:

"My doctors never knew what is was, I was prescribed slimming pills aged 18 years ald but didn't like the side effects and my legs never changed."

Mr 5 was referred to the brophochem data and anesoed when due was 20 years did, where die was preserblee compression light. Here Joga lecourie legge following by which time due uses unterruter compression throughout. In 2011, Mr 5 had a lengthostenargody sens, which continent algab lengthostena in the right log but models bacedom wereal.

The second se

#### she handsges, they kept unravelling and they were to big that I had to wear 'size 8 errors', not a good look when you are a size 6 shoe.'

Mes'S found bundaging challenging in that they did not say in place, so she decided to swap them for casystrap. Where .. What a difference, I put on the four/ lower leg easympap and they were so much more comfortable, stayed secure and I could user normal footwear on top. I did believe this has had massive improvement to my meeting / recovery. I have warn the swaps daily for 6 weeks now and I still sleep in compression but I've gone back to wearing my Haddenham Veni compression. Hanse Klinik do advise that it will take up to 4 months for swelling to go down, but I think my legs look great already-only 6 weeks and 2 days after surgery! None I feel completely amazing. I'm the happiest Pre ever been in my life. I feel like a proper girl for the first time in my life. I love wearing dresses and thirts, my legs feel light and free. I'm happy wearing compression, I like the support it gives me and it's a bit like a comfort blanker. I continue to surim, attending local pools and do not feel conscious about people looking at my legal My next aim is to join a netball team, as I have not worn a netball skirs since I was at school because I was so self-conscious of my legs, but now I can wear a shire with pride and show off my atunning legs Per

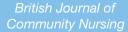




Figure 9. Mrs W wearing easywrap

my lymphoedems by handaging my log for me regularly and massages my log, hips, stomach and back every day?

During 2016, Mrs W attended LymphVision for mapping, which confirmed dermal re-sound and backflow in the affected leg trabo confirmed that lymph was draining into

## COST EFFECTIVE AND TIME SAVING

## 78 Hours Nursing Time

7 Hours Nursing Time

A simple and cost effective alternative solution for wound healing...

## Bandaging

78 Appointments 3x 1 hour appointments per week

Cost of Bandages £1,935.96

Cost of Nursing Time £1,872.00

Total Cost £3,807.96 Total Cost £604.92



### **13 Appointments** 1x 0.5 hour appointment per week


Cost of easywrap £292.92


Cost of Nursing Time £312.00

Cost comparison based over 6 months, treating bilateral legs and does not include travel time, primary wound dressings or mileage expenses. Cost comparisons based on those used in: https://hadhealth.com/assets/articles/BJCN\_24(4)\_Haddenham\_PF\_web.pdf \*prices shown based on 2019 Drug Tariff pricing. the full article

**SCAN** 



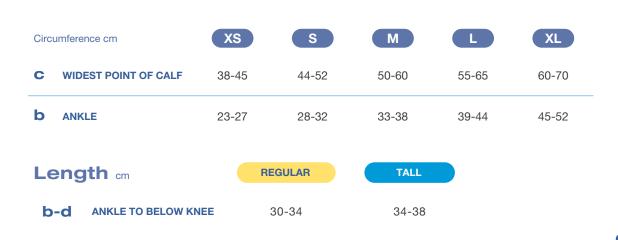

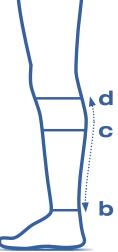




	light 20-30 mmHg			<b>stro</b> 30-40	
	REGULAR	TALL		REGULAR	TALL
XS	400-4917	400-4966		400-5518	400-5567
S	400-4925	400-4974		400-5526	400-5575
М	400-4933	400-4982		400-5534	400-5583
L	400-4941	400-4990		400-5542	400-5591
XL	400-4958	400-5005		400-5559	400-5609
XS	408-9397	408-9355		408-9009	408-8951
S	408-9405	408-9389		408-8985	408-8969
M	408-9413	408-9363		408-8993	408-8944
C	408-9421	408-9348		408-9025	408-8977
XL	408-9439	408-9314		408-9041	408-8928




Circumference cm	XS	S	М	L	XL
a <sup>1</sup> MID-FOOT	22-24	25-27	28-30	31-34	35-40
Length cm					
a-heel REGULAR	14.5-16.5	16-18	17.5-19.5	19-21	21-23
LONG	16.5-18.5	18-20	19.5-21.5	21-23	23-25

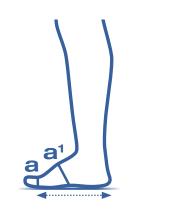


	light 20-30 mmHg			strong 30-40 mmHg			
	REGULAR	LONG		REGULAR	LONG		
XS	400-4867	400-4818		400-5468	400-5419		
S	400-4875	400-4826		400-5476	400-5427		
M	400-4883	400-4834		400-5484	400-5435		
C	400-4891	400-4842		400-5492	400-5443		
XL	400-4909	400-4859		400-5500	400-5450		
XS	408-9553	408-9447		409-0064	409-0023		
S	408-9546	408-9454		409-0049	409-0056		
M	408-9538	408-9488		409-0072	409-0031		
L	408-9520	408-9496		408-9090	408-9074		
XL	408-9512	408-9504		408-9116	408-9082		



# THE GH WITH KNEE

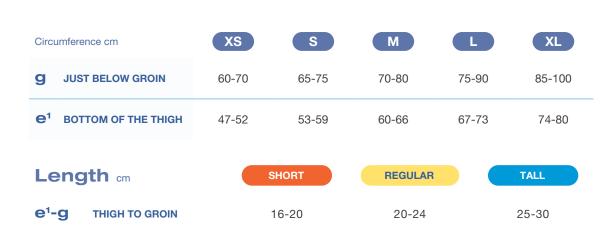

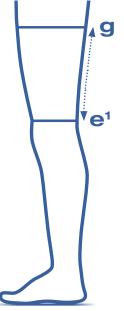


		light 20-30 mmHg			9	Strong 30-40 mmHg	I
	SHORT	REGULAR	LONG		SHORT	REGULAR	LONG
XS	400-5062	400-5013	400-5112		400-5666	400-5617	400-5716
S	400-5070	400-5021	400-5120	-	400-5674	400-5625	400-5724
М	400-5088	400-5039	400-5138	-	400-5682	400-5633	400-5732
L	400-5096	400-5047	400-5146	-	400-5690	400-5641	400-5740
XL	400-5104	400-5054	400-9816	_	400-5708	400-5658	400-5757
XS	408-9223	408-9231	408-9124		408-8886	408-8779	408-8712
S	408-9215	408-9256	408-9132		408-8910	408-8795	408-8746
M	408-9207	408-9272	408-9140		408-8902	408-8811	408-8753
L	408-9181	408-9280	408-9165		408-8845	408-8787	408-8738
XL	408-9199	408-9298	408-9173		408-8860	408-8837	408-8704



# A-C LINERS


Suitable for use with easywrap foot and leg garments. Please select the appropriate size for your easywrap garments.

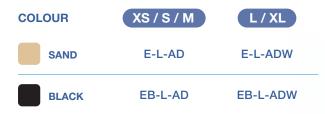





Suitable for use with easywrap foot, leg and thigh garments. Please select the appropriate size for your easywrap garments.

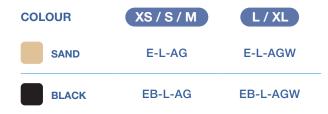



no compression

# FUSION

easywrap fusion is a liner and foot piece in one, and makes the perfect partner for an easywrap leg piece. The liner provides compression to the foot, and half-compression at the ankle, offering a low-profile, smart alternative to regular foot wraps.

half-compression zone
(beneath the last band of easywrap leg piece)


---- compressive foot

							c
Circumference cm	XS	S	М	L	XL		
C WIDEST CALF	≤ 45	≤ 50	≤ 60	≤ 65	≤ 70		
<b>b</b> ANKLE	19-24	25-29	30-34	35-40	41-46	a	b
a <sup>1</sup> MID-FOOT	19-24	25-29	30-34	35-40	41-46		

COLOUR	COMPRESSION	XS	S	М	L	XL
	Light 20-30mmHg	415-3110	415-3128	415-3136	415-3144	415-3151
SAND	Strong 30-40mmHg	415-3243	415-3235	415-3276	415-3268	415-3250
	Light 20-30mmHg	415-3219	415-3185	415-3193	415-3201	415-3227
BLACK	Strong 30-40mmHg	415-3284	415-3292	415-3318	415-3326	415-3300



Circumference cm	S	М		XL
g axilla	22-31	29-39	32-45	36-50
e ELBOW CREASE	20-27	25-34	30-40	32-43
C WRIST CREASE	14-18	16-21	19-25	19-25
Length cm	SHORT		REGULAR	TALL
C-g AXILLA TO WRIST	40-44		44-48	48-52

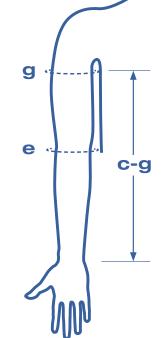

		left 20-30 mmHg				right 20-30 mmHg	
(	SHORT	REGULAR	LONG		SHORT	REGULAR	LONG
S	400-5187	400-5245	400-5302	-	400-5153	400-5211	400-5278
М	400-5195	400-5252	400-5310	-	400-5161	400-5229	400-5286
L	400-5203	400-5260	400-5328		400-5179	400-5237	400-5294
XL	405-8236	405-8194	405-8186	-	405-8202	405-8210	405-8228
S	408-8373	408-8290	408-8241	-	408-8415	408-8332	408-8258
М	408-8381	408-8308	408-8225	-	408-8423	408-8340	408-8266
C	408-8399	408-8316	408-8233		408-8431	408-8357	408-8274
XL	408-8407	408-8324	408-8217		408-8449	408-8365	408-8282

# HAND

Circumference cm	XS	S	М	L	
<b>b</b> PALM AT FOLD OF THUMB	<19	19-21	21-23	>24	<b>b</b>
<b>a</b> PALM AT BASE OF FINGERS	<18	18-20	20-22	>22	a

	left 20-30 mmHg	right 20-30 mmHg
	REGULAR	REGULAR
XS	400-5377	400-5369
S	400-5385	400-5336
М	400-5393	400-5344
L	400-5401	400-5351
XS	408-9603	408-9595
S	408-9637	408-9587
М	408-9629	408-9579
Ľ	408-9611	408-9561



EWB-002 ver.04

Tel: 01844 208842 Email: sales@hadhealth.com Website: www.hadhealth.com