

## INFORMATION

# KLEBI SKIN ADHESIVE

### Description:

Water soluble liquid adhesive to maintain position of hosiery, orthopaedic aids, bandages and other textile articles

Non-slip fixing of textile articles; skin compatible; non-toxic; water soluble; non-staining; easy to wash off



### Directions for use:

Remove cap; simply roll on to skin and press article into place. For support stockings – pull up stockings; turn down topband; apply adhesive around leg several times; pull up top band and press into place; replace cap tightly and store in an upright position.

During use: discontinue use if irritation occurs

After use: remove adhesive with water

### Components:

Cap and ball are polypropylene (pp 05); bottle is high density polypropylene (pe 2); adhesive composition is a trade secret under the hazard communication standard 29 cfr 1910.1200.

Nb: components can be found in current retail cosmetics and food products.